



FOUR STAR

DENTAL SERVICE, INC.

— "Quality Products. Value Prices." —

PHONE: (877)-CEREC2U

FAX: (386) 310-7219

WWW.FOURSTARDENTALSERVICE.COM

How to Force a Download to Your Milling Unit

1. Turn on the acquisition unit and open the Cerec 3D program.
2. On the top menu bar is a tab for devices, select milling unit.
3. If you see a non-connected milling unit, turn off the power switch located on the back of the milling unit.
4. Open the front door where the water reservoir is located.
5. Press the button towards the right of the unit on the metal plate and hold it down.
6. Turn the power switch on the back of the unit back on.
7. Count 17 seconds and then release the metal button.
8. Look for a message that says "downloading software".
9. You may have to repeat this procedure several times before it will work.

Feel free to contact us at (877) 237-3228 for any assistance with this process.