



**How to Set-up Your Cerec:**

1. Remove the back panel of the acquisition unit.
2. Place the monitor on the unit and secure with the black thumb screws.
3. Connect the round silver collar and blue monitor cable.
4. Insert the camera (ensure it is threaded properly).
5. Connect the power cord to the unit (do NOT plug it in or turn it on yet).
6. Connect the wireless receiver to the back of the milling unit and tighten the two small screws.
7. Add 25 mm of milling oil and fill water reservoir up with distilled water.
8. Verify burs are installed in each side of the milling unit.
9. If you have 220 Voltage, you will need to change the power supply setting on your acquisition unit now. At the bottom right of the power cord is a small slot where you can see the power supply settings. Take a small screw driver or knife and push in the tab on the left and the assembly will pop out. Turn the cube to 220 V/AC. The milling unit has an automatic voltage regulator built into it.
10. Insert the power plug and turn on each unit.
11. You're ready to begin using your Cerec!