

Update Description

The service pack CEREC SW 4.2.1 updates existing CEREC SW 4.2 installations.

The service pack contains the following changes:

■ **Driver problem affecting the Bluecam and inEos Blue corrected**

In some cases the installation of the new software altered the driver settings for the Bluecam and inEos Blue in such a way that it was no longer possible to acquire images. (The software reports that the camera is being used by another application.) This problem is permanently corrected with service pack 4.2.1

■ **User account control**

Changes to the user account settings in Windows can lead to complications (e.g. failure to recognize the camera or milling unit; errors during the import and export of data). With service pack 4.2.1 the user account setting has no impact on the CEREC SW regardless of the specific setting.

■ **Message if at least one license drive is not up-to-date**

The firmware version of all connected license drives will be automatically checked. If at least one license drive has an outdated firmware the software will request the user to update it.

■ **Optimization of the Omnicam scan process**

The optimization results in a faster and more tolerant scanning process with the CEREC Omnicam.

■ **Fixed error if restoration parameters are opened with an Italian language setting**

■ **Fixed error sometimes occurring while drawing the margin**

■ **Optimized blocking out of inlay preparations with strong undercuts for Omnicam**

An existing CEREC SW 4.2 installation is required for this service pack.

We recommend all CEREC SW 4.2 users to install this service pack. .