



Milling Unit Purging Procedure

WARNING! Failure to follow these instructions will result in damage to your unit during shipping.

1. From the PC, select *Start*.
2. Select *Programs*.
3. Scroll over *CEREC*.
4. Select *Service*.
5. You will be prompted to enter a password, select *Maintenance Only*.
6. Make sure milling unit is turned on. Select *Refresh* until it says *Milling Unit Ready*.
7. Select *Milling Unit*.
8. Dialog box will appear. Turn on the pump to empty the cooling coil. Please follow instructions in manual for your particular milling unit. **Ensure the reservoir milling tank in the unit is empty before you begin the process.**
9. Select *Start Now* (or *Yes*) and let it run for approximately 2-3 minutes or until it stops bubbling.
10. Select *Stop Now* and then empty the water tank once again.
11. Done!

Contact Sirona's technical support at (800) 659-5977 with any questions in regards to this process.